

CovaWound™ Silicone

Self-Adherent Soft Silicone Foam Dressing with Border

WHAT IS IT?

CovaWound™ Self-Adherent Soft Silicone Foam Dressing with Border is a 5-layer, atraumatic dressing that absorbs exudate, maintains a moist wound healing environment, and prevents epidermal stripping and/or pain upon removal.

HOW DOES IT WORK?

CovaWound™ Self-Adherent Soft Silicone Foam Dressing with Border has a soft silicone wound contact layer which facilitates application and retention of the dressing to intact skin, but will not stick to the surface of a wound or cause trauma to delicate new tissue upon removal. The centre of the dressing consists of highly absorbent polyurethane foam, a non-woven spreading layer, and absorbent fibres designed to lock-in exudate and prevent the wound bed from drying out or the surrounding skin from becoming macerated. The top layer of the dressing is a vapour permeable polyurethane membrane that acts as a barrier to liquid and microorganisms.

WHEN DO I USE IT?

CovaWound™ Self-Adherent Soft Silicone Foam Dressing with Border is indicated for the management of exuding wounds including:

- Leg and foot ulcers
- Pressure ulcers
- Traumatic and surgical wounds
- Superficial and partial thickness burns
- CovaWound™ Self-Adherent Soft Silicone Foam Dressing with Border can be used under compression bandaging

Please refer to package insert for full instructions.



Reference Number	Size (in/cm)	Dressings per Carton	Cartons per Case	HCPCS
1017575	3in x 3in 7.5cm x 7.5cm	10	20	
1011010	4in x 4in 10cm x 10cm	10	20	
1011212	5in x 5in 12.5cm x 12.5cm	10	20	
1011515	6in x 6in 15cm x 15cm	10	20	
1011520	6in x 8in 15cm x 20cm	10	20	
Sacrum 1011818	7in x 7in 18cm x 18cm	10	20	
Heel 1011220	5in x 8in 12.5cm x 20cm	10	20	

SUGGESTED WOUND TISSUE TYPE:

NECROTIC (After debridement)	✓
SLOUGHY	✓
GRANULATING	✓
EPITHELIALISING	✓
INFECTED	—

ABSORPTION LEVEL: HEAVY

