

CovaWound™ Alginate

Alginate Dressing



WHAT IS IT?

CovaWound™ Alginate is a primary wound dressing made from the calcium salt of alginic acid rich in mannuronic acid. It is presented as a flat non-woven pad for application to open wounds, as well as a rope for packing of wound cavities.

HOW DOES IT WORK?

When CovaWound™ Alginate is in contact with serum, wound exudate, or solutions containing sodium ions, the insoluble calcium alginate is partially converted to the soluble sodium salt, and a soft, hydrophilic, gas permeable gel is rapidly produced, which conforms to the contours of the wound and provides a micro-environment that facilitates wound healing.

WHEN DO I USE IT?

CovaWound™ Alginate may be applied to moderate to heavy exuding wounds including:

- Partial thickness burns
- Donor sites, leg, pressure, arterial, diabetic and venous stasis ulcers
- Cavity wounds
- Post-surgical incisions
- Trauma wounds
- Most other granulating wounds

Please refer to package insert for full instructions.

Reference Number	Size (in/cm)	Dressings per Carton	Cartons per Case	HCPCS
3000505	2in x 2in 5cm x 5cm	5	40	A6196
3001010	4in x 4in 10cm x 10cm	5	20	A6196
3001020	4in x 8in 10cm x 20cm	5	20	A6197
3001520	6in x 8in 15cm x 20cm	5	20	
Rope 3000230	1in x 12in 2cm x 30cm	5	20	A6199
Rope 3000240	1in x 16in 2cm x 40cm	5	20	

SUGGESTED WOUND TISSUE TYPE:

NECROTIC (After debridement)	✓
SLOUGHY	✓
GRANULATING	✓
EPITHELIALISING	✓
INFECTED	—

ABSORPTION LEVEL: HEAVY

