

CovaWound™ + ColActive®

Dressing Solution Guide

WOUND TISSUE TYPE	APPEARANCE	COVAWOUND + COLACTIVE SOLUTION
NECROTIC	Dry black / brown tissue (eschar) composed of dead, dehydrated epidermis and underlying tissue.	<ul style="list-style-type: none"> • Surgical or sharp debridement. ✓ Autolytic debridement with hydrocolloid dressing or hydrogel.  <p>CovaWound Hydrocolloid Hydrocolloid Dressing</p> <ul style="list-style-type: none"> ✓ Non or low adherent dressing with a dry secondary dressing.  <p>CovaWound Silicone Self-Adherent Soft Silicone Foam</p>  <p>CovaWound Silicone Self-Adherent Soft Silicone Foam with Border</p>  <p>CovaWound Foam Non-Adherent Foam</p>  <p>CovaWound Foam Non-Adherent Foam with Adhesive Border</p> <ul style="list-style-type: none"> ✓ Collagen matrix dressing to re-initiate healing if applicable.  <p>ColActive Plus Collagen Matrix Dressing</p>  <p>ColActive Plus Ag Collagen Matrix Dressing with Silver</p>
SLOUGHY	Moist, soft, stringy white to yellow coloured necrotic tissue.	<ul style="list-style-type: none"> • Surgical or sharp debridement. ✓ Autolytic debridement with hydrocolloid dressing or hydrogel.  <p>CovaWound Hydrocolloid Hydrocolloid Dressing</p> <ul style="list-style-type: none"> ✓ If heavy exudate is present: Foam, alginate or super absorbent may be appropriate. Alginate Rope if packing is required.  <p>CovaWound Silicone Self-Adherent Soft Silicone Foam</p>  <p>CovaWound Silicone Self-Adherent Soft Silicone Foam with Border</p>  <p>CovaWound Foam Non-Adherent Foam</p>  <p>CovaWound SuperAbsorbent SuperAbsorbent Dressing</p>  <p>CovaWound Alginate Alginate Dressing</p>  <p>CovaWound Alginate Alginate Dressing (Rope)</p> <ul style="list-style-type: none"> ✓ Re-initiate healing with collagen matrix dressing if applicable.  <p>ColActive Plus Collagen Matrix Dressing</p>  <p>ColActive Plus Ag Collagen Matrix Dressing with Silver</p> <ul style="list-style-type: none"> ✓ Negative pressure wound therapy if applicable.  <p>ColActive Transfer Wound Contact Layer</p>
GRANULATING	Firm, red to pale pink tissue with a bumpy or 'cobblestone' appearance.	<ul style="list-style-type: none"> ✓ Moderate to heavy exudate: Foam, alginate or super absorbent may be appropriate. Alginate Rope if packing is required.  <p>CovaWound Foam Non-Adherent Foam</p>  <p>CovaWound SuperAbsorbent SuperAbsorbent Dressing</p>  <p>CovaWound Alginate Alginate Dressing</p>  <p>CovaWound Alginate Alginate Dressing (Rope)</p>

CovaWound™ + ColActive®

Dressing Solution Guide

WOUND TISSUE TYPE	APPEARANCE	COVAWOUND + COLACTIVE SOLUTION
GRANULATING (CONTINUED)	Firm, red to pale pink tissue with a bumpy or 'cobblestone' appearance.	<ul style="list-style-type: none"> ✓ Low exudate: Hydrocolloid, semi-permeable film dressing. <ul style="list-style-type: none">  CovaWound Hydrocolloid Hydrocolloid Dressing  CovaWound Foam Non-Adherent Foam with Adhesive Border ✓ Collagen matrix dressing to re-initiate healing if applicable. <ul style="list-style-type: none">  ColActive Plus Collagen Matrix Dressing  ColActive Plus Ag Collagen Matrix Dressing with Silver ✓ Negative pressure wound therapy if applicable. <ul style="list-style-type: none">  ColActive Transfer Wound Contact Layer
EPITHELIALISING	Pale pink to white tissue spreading from the wound edges or as islands within the wound.	<ul style="list-style-type: none"> ✓ Moderate to heavy exudate: Foam, alginate or super absorbent may be appropriate. Alginate Rope if packing is required. <ul style="list-style-type: none">  CovaWound Foam Non-Adherent Foam  CovaWound SuperAbsorbent SuperAbsorbent Dressing  CovaWound Alginate Alginate Dressing  CovaWound Alginate Alginate Dressing (Rope) ✓ Low exudate: Hydrocolloid, semi-permeable film dressing. <ul style="list-style-type: none">  CovaWound Hydrocolloid Hydrocolloid Dressing  CovaWound Foam Non-Adherent Foam with Adhesive Border ✓ Collagen matrix dressing to re-initiate healing if applicable. <ul style="list-style-type: none">  ColActive Plus Collagen Matrix Dressing  ColActive Plus Ag Collagen Matrix Dressing with Silver
INFECTED	Any of the above tissue types with associated signs of infection, including purulent discharge, surrounding erythema, pain, etc.	<ul style="list-style-type: none"> • Systemic antibiotics with suitable dressings depending on wound characteristics mentioned. ✓ Silver dressings. <ul style="list-style-type: none">  ColActive Plus Ag Collagen Matrix Dressing with Silver  CovaWound Alginate Ag Alginate Dressing with Silver  CovaWound Alginate Ag Alginate Dressing with Silver (Rope)