


CovaWound™

Alginate **Ag**

Alginate Dressing with Silver

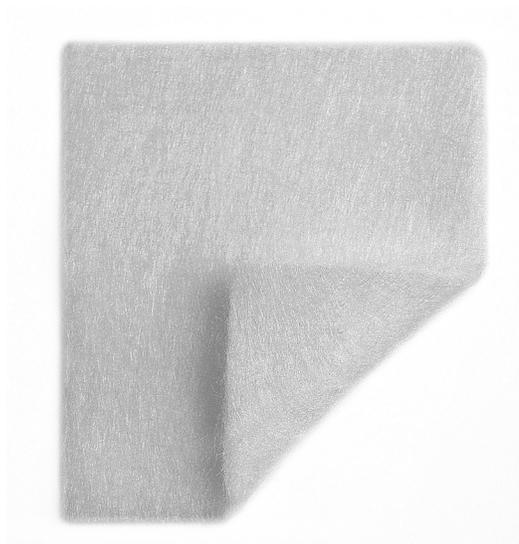

WHAT IS IT?

CovaWound™ Alginate Dressings with Silver are soft, comfortable wound dressings with a high mannuronic acid content.

HOW DOES IT WORK?

The silver-impregnated calcium alginate fibers, when in contact with wound exudate or blood, form a gel which creates a moist wound healing environment for optimal wound healing. The silver ions protect the dressing from a range of microorganisms.

WHEN DO I USE IT?

CovaWound™ Alginate Dressings with Silver can be used for the management of wounds which are moderate and heavily exuding, including:

- Superficial or deep wounds, such as pressure sores, arterial ulcers, venous lower leg ulcers, diabetic ulcers,
- Post-operative wounds

Please refer to package insert for full instructions.

Reference Number	Size (in/cm)	Dressings per Carton	Cartons per Case	HCPCS
3100505	2in x 2in 5cm x 5cm	10	20	
3101010	4in x 4in 10cm x 10cm	10	20	
3101020	4in x 8in 10cm x 20cm	5	20	
Rope 3100230	1in x 12in 2cm x 30cm	5	20	

SUGGESTED WOUND TISSUE TYPE:

NECROTIC (After debridement)	✓
SLOUGHY	✓
GRANULATING	✓
EPITHELIALISING	✓
INFECTED	✓

ABSORPTION LEVEL: HEAVY

